



Skin and Wound Product Information Sheet

Zipzoc	
Classification	Contact Layer: Zinc Impregnated Stocking
Key Points	<ul style="list-style-type: none"> • 20% zinc oxide impregnated stocking • Preservative free
Indications	<ul style="list-style-type: none"> • For chronic eczema and dermatitis where occlusion is indicated • May be used in conjunction with appropriate compression therapy
Precautions	<ul style="list-style-type: none"> • Discontinue use if redness, itching or if wound present, deterioration of the wound occurs and notify Wound Clinician, NP or Physician • If using Acticoat Flex3 or Acticoat Flex7 for wound care, then use a gauze dressing as an interface to prevent the paste's emulsifier from coming in contact with the dressing
Contraindications	<ul style="list-style-type: none"> • Do not use for clients with known sensitivity to zinc oxide or other product ingredients • Do not use in the presence of uncontrolled clinical infection • Do not use for clients with arterial compromise/ulcers
Size and formats	<ul style="list-style-type: none"> • Stocking <ul style="list-style-type: none"> ▪ 14 cm x 86 cm <div style="text-align: right;">  </div>
Application Directions	Rationale
Wash or shower legs with warm water and pat dry.	To remove dead skin.
To Apply	
Slip foot into stocking then draw it up from base of toes to below the knee; fold any excess stocking below the knee.	
Smooth the stocking to remove any wrinkles and to mould it to the shape of the leg.	Removing the wrinkles will help to prevent areas of potential pressure.
Wrap the leg with kling in a loose spiral; secure with tape or mesh stockingnet.	To protect client's clothing/ environment from the zinc.
To Remove	
Carefully cut the stocking off.	Lift the stocking away from the skin while cutting to avoid cutting the underlying skin.
Frequency of Dressing Change	
Change at least once a week or when stocking slips.	
Expected Outcome	
Eczema or dermatitis is resolved within 2 weeks.	
For further information, please contact your Wound Clinician.	