



Skin and Wound Product Information Sheet

Adapt Adhesive Remover	
Classification	Adhesive Remover
Key Points	<ul style="list-style-type: none"> • Silicone-based, alcohol-free solution formulated to dissolve adhesives e.g., acrylic- based, rubber-based, and hydrocolloid-based residues from the skin. • As the product contains no alcohol; may be used on broken/denuded skin. • Product quickly evaporates; Spray does not contain Chlorofluorocarbons (CFCs) • Latex-free
Indications	<ul style="list-style-type: none"> • Use on skin to remove residue from adhesive products e.g. dressings, tapes, wafers, ostomy flanges hydrocolloids • Use to assist with the removal of adhesive dressings, tapes, wafers, ostomy flanges, hydrocolloids • May be used for all ages including neonates
Precautions	N/A
Contraindications	<ul style="list-style-type: none"> • Do not use directly on wounds, burns or stomas
Formats & Sizes	<ul style="list-style-type: none"> • Wipe – Universal Adhesive Remover • Spray – Medical Adhesive Remover 50ml <div style="text-align: center;"> </div>
Application Directions	
<p>To remove adhesive residue:</p> <ul style="list-style-type: none"> • cleanse the area with the wipe or lightly apply the spray to the area • cleanse the area to remove any adhesive residue with at least room temperature water or normal saline and pat dry 	Rationale
<p>To assist with removing of adhesive product:</p> <ul style="list-style-type: none"> • gently lift a corner of the product to be removed • wipe the adhesive remover between the skin and the product then gently peel back the product while continue to wipe • cleanse the area to remove any adhesive residue with at least room temperature water or normal saline and pat dry 	
Expected Outcome	
Adhesive will be removed leaving the skin healthy and intact.	
For further information, please contact your NSWOC/Wound Clinician.	