


## Product Information Sheet

Adapt Adhesive Remover	
<b>Classification</b>	<b>Adhesive Remover</b>
<b>British Columbia Practice</b>	<ul style="list-style-type: none"> <li>The British Columbia Perinatal Services and the British Columbia Provincial Nursing Skin &amp; Wound Committee have determined Adapt Adhesive Remover for neonates in:               <ul style="list-style-type: none"> <li>NICUs with a corrected gestational age greater than 31 weeks + 6 days.</li> <li>Maternity/pediatric units who are less than 28 days in age.</li> </ul> </li> </ul>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>Silicone-based, latex-free, alcohol-free solution formulated to dissolve adhesives, (e.g., acrylic-based, rubber-based), and hydrocolloid-based residues from the skin.</li> <li>Product does not causing stinging as the product does not contain alcohol; may be used on broken/denuded skin.</li> <li>Product quickly evaporates.</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>Use for all ages including neonates (see practice statement above):               <ul style="list-style-type: none"> <li>To assist with the removal of adhesive products, (e.g., tapes, ostomy flanges, hydrocolloids).</li> <li>To remove adhesive product residue from skin.</li> </ul> </li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>Extremely flammable; keep away from sources of ignition.</li> <li>Discontinue use if irritation noted.</li> <li>Do not use on around eyes, ears nose, mouth or vagina.</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>Do not use directly on wounds, burns or stomas.</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>Wipe.</li> <li>Spray.</li> </ul> 

Directions	Rationale / Key Points
<b>Application to Remove a Product</b>	
Do not open the package until ready to use. Ensure area is dry.	Opening the package immediately on use will ensure the wipe does not dry out.
Gently lift a corner of the product to be removed and wipe or spray the adhesive remover between the skin and the product. Slowly peel back the product while continuing to wipe.	
Cleanse the area to remove any adhesive residue with warm dampened cloth/wipe or normal saline/sterile water-soaked gauze. Pat to dry the area.	
Ensure skin is dry before applying tape, ostomy flange, etc.	
<b>Application to Remove Adhesive Residue</b>	
Gently wipe or lightly spray the adhesive remover over area.	
Cleanse the area to remove any adhesive residue with warm dampened cloth/wipe or normal saline/sterile water-soaked gauze. Pat to dry the area.	
Ensure skin is dry before applying tape, ostomy flange, etc.	
<b>Expected Outcomes</b>	
Adhesive will be removed leaving the skin healthy and intact.	If product does not perform as expected, notify NSWOC/Wound Clinician and then consider submitting a <a href="#">Supply Chain Product Concern Form</a> .
<b>For further information please contact NSWOC/Wound Clinician</b>	