



## Skin and Wound Product Information Sheet

<b>Adaptic for Digits</b>	
<b>Classification</b>	<b>Contact Layer: Digits</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• A silicone coated contact layer with a double-coiled tubular bandage designed to fit fingers and toes</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• For minor to severe injuries to fingers and toes</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• N/A</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Do not use for clients with known sensitivity or allergy to silicone</li> <li>• Do not use during Hyperbaric Oxygen therapy procedure</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>• Digit dressing                             <ul style="list-style-type: none"> <li>▪ Finger 2.0 cm (small)</li> <li>▪ Finger 2.4 cm (medium)</li> <li>▪ Finger 2.8 cm (large)</li> <li>▪ Finger 3.0 cm (x-large)</li> <li>▪ Toe 2.8 cm</li> </ul> </li> </ul> <div style="text-align: right;">  </div>
<b>Application Directions</b>	
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing .
Choose a size of digit dressing to fit the finger(s) or toes.	
<b>To Apply</b>	
Remove shiny backing strip on the non-adhering layer.	Single layer allows passage of exudate through to cover dressing.
Place non-adhering layer over wound and smooth into place.	
If required, apply appropriate cover dressing to maintain a moisture-balanced wound environment.	The choice of an additional cover dressing will depend upon the amount of exudate expected.
Roll down lower of tube bandage over the non-adhering layer and cover dressing if being used. Then apply the top ring to form a double-layer tube dressing.	
<b>To Remove</b>	
Carefully lift away the dressing and discard.	To avoid trauma to the wound bed.
<b>Frequency of Dressing Change</b>	
Will depend upon the amount of exudate. Adaptic Digit may be left in place for up to 7 days.	
<b>Expected Outcome</b>	
Contact layer does not adhere to wound.	
<b>For further information, please contact your Wound Clinician.</b>	

### How to apply ADAPTIC\* Digit Non-Adhering Digit Dressing

