

## Skin and Wound Product Information Sheet

<b>Allevyn Life</b>	
<b>Classification</b>	<b>Cover Dressing: Silicone Foam</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• Multilayered dressing with a silicone adhesive contact layer, hyper-absorbent core, absorbent hydrocellular foam and a shower-proof exudate strike-proof protective top layer</li> <li>• Wide border and unique shape</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• For wounds with moderate to large amount of exudate</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• N/A</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Do not use with oxidizing agents such as hypochlorite solutions (Eusol) or hydrogen peroxide</li> <li>• Do not use if redness or sensitivity occur</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>▪ Bordered dressing                             <ul style="list-style-type: none"> <li>▪ 10.3 x 10.3 cm</li> <li>▪ 12.9 x 12.9 cm</li> <li>▪ 15.4 x 15.4 cm</li> <li>▪ 21 x 21 cm</li> <li>▪ 17.2 x 17.5cm(sacrum)</li> <li>▪ 21.6 x 23cm(sacrum)</li> </ul> </li> </ul> <div style="text-align: right;">  </div>
<b>Application Directions</b>	
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply a skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
<b>To Apply</b>	
Choose a dressing size that will ensure that the dressing extends at least 2cm beyond wound edge.	Incorrect sizing will adversely affect the dressing absorption functionality.
Place the dressing directly over the center of the wound ensuring the white side is applied to the wound.	
Apply directly on the wound as a primary dressing or in combination with another wound product as a secondary cover dressing.	Needs to be in contact with the wound to be most effective.
<b>To Remove</b>	
Gently lift the border to remove the dressing.	To minimize trauma to the peri-wound skin.
<b>Frequency of Dressing Change</b>	
Will depend upon the amount of exudate; change when exudate extends to 75% of the foam border. Can be left on up to 7 days.	The absorbed exudate is clearly visible through the pink backing of the dressing
<b>Expected Outcome</b>	
Exudate is managed with no peri-wound skin maceration.	
<b>For further information, please contact your Wound Clinician.</b>	