




## Skin and Wound Product Information Sheet

<b>Debrisoft Lolly</b>	
<b>Category</b>	<b>Debrider / Mechanical</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>Lolly with break-resistant handle and cleansing pad comprised of polyester fibres used to gently lift, bind, and remove slough and debris (including biofilm) from the wound bed.</li> <li>When used with a gentle scrubbing motion on the wound bed, the fibres remove and retain debris, biofilm, and exudate with minimal or no pain and no damage to viable tissue.</li> <li>Used to cleanse wounds with depth, undermining, sinus tracts or tunnels.</li> <li>May be used as the only method of debridement or in combination with other methods (e.g., autolytic debridement).</li> <li>Treatment can be repeated until wound bed or skin is clean and free from debris.</li> <li>Use <a href="#">Cutimed Debriclean</a> pads for superficial wounds or wounds with minimal depth.</li> </ul>
<b>Indications</b>	Use under the direction of an NSWOC, Wound Clinician, Physician or Nurse Practitioner (NP) for: <ul style="list-style-type: none"> <li>Wounds with depth, undermining, sinus tracts or tunnels which contain slough and/or non-adherent or lightly-adherent soft necrotic tissue.</li> <li>Wounds with depth, undermining, sinus tracts or tunnels which have biofilm.</li> <li>Hematomas with non-adherent or lightly-adherent debris.</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>May be used on infected wounds in conjunction with appropriate clinical treatment (e.g., antibiotic therapy, antimicrobial dressings).</li> <li>If required, soluble Lidocaine gel is compatible with Debrisoft Lolly. A Physician/NP order is required unless there is Health Authority approval for NSWOC or Wound Clinician to order.</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>Wounds with dry or adherent necrotic slough or eschar. Soften using autolytic debridement before using the Debrisoft Lolly.</li> <li>Untreated infected wounds.</li> <li>Wounds with exposed bone or tendon.</li> <li>Clients with allergy or sensitivity to the product's components.</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>Lolly – monofilament head 5 x 1.9 cm, total length 15.5 cm</li> </ul> <div style="text-align: center;">  </div>
<b>Directions for Use</b>	
Do not cut pad.	To decrease the risk of loose fibres.
<b>To Use</b>	
For dry necrosis or dense slough, utilize other debridement methods prior to using Debrisoft Lolly.	Debrisoft may not be as effective on thick, hard eschar or thick, tenacious slough.
Cleanse/ irrigate wound with sterile normal saline or agency approved wound cleanser.	To remove loose debris. Can be used with all common cleansing solutions, including antiseptics and antimicrobial cleansers.
If wound is large enough use Cutimed Debriclean Pad prior to using Lolly for less accessible spaces.	Refer to <a href="#">Cutimed Debriclean</a> product information sheet.
Thoroughly moisten soft, fleecy part of lolly with minimum of 15 ml normal saline, water or other agency approved wound cleanser.  Allow excess fluid to drip off prior to applying to wound. Do not wring out.	It may be easier to fill section of dressing tray with wound cleansing solution and then soak Lolly.
Grip the Lolly by its handle.	



## Skin and Wound Product Information Sheet

Directions for Use	Rationale / Key Points
<p>Insert soft, fibre head of Lolly into wound, undermining, sinus tract or tunnel. <b>DO NOT</b> insert into a wound, sinus or tunnel smaller than the fibre head.</p> <p>Gently with controlled pressure, use the soft fleecy part of the product in circular motion or sweeping strokes to disrupt and remove non-viable tissue and debris from wound.</p> <p>Use a different Lolly when fibres have collected sufficient slough.</p>	<p>Do not insert Lolly further than handle allows to ensure easy removal.</p> <p>Use as much pressure as is tolerated by client.</p>
<p>Dispose of Lolly after use. Do not reuse.</p> <p>Use a new Lolly for each separate wound.</p>	<p>Decrease risk of cross-contamination.</p>
<p>If periwound skin shows signs of hyperkeratosis or flakey debris cleanse this area with a Cutimed Debriclean Pad.</p>	<p>Refer to <a href="#">Cutimed Debriclean</a> product information sheet.</p>
<p>Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser.</p>	<p>To remove debris and necrotic tissue.</p>
<p>Cleanse and dry periwound skin.</p>	
<p>Apply dressing as per wound treatment plan.</p>	
Expected Outcome	
<p>Wound bed is clean and free of debris.</p> <p>Product performs as expected.</p>	<p>If product does not perform as expected notify NSWOC/Wound Clinician and consider submitted a Product Concern form.</p>
<p><b>For further information, please contact: NSWOC or Wound Clinician</b></p>	