



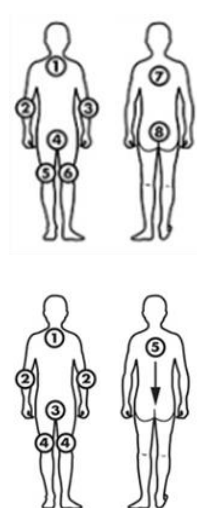


Skin and Wound Product Information Sheet

Essential Bath Cleansing Washcloths	
Classification	Cleanser: Prepared Cloth
Key Points	<ul style="list-style-type: none"> Pre-moistened, medium weight washcloth containing mild cleansers, purified water, and skin moisturizers with aloe and Vitamin E. Thick, hypoallergenic, and ultra-soft cloths. Fragrance-free. pH balanced. Rinse free.
Indications	<ul style="list-style-type: none"> For bathing patients/clients/residents.
Precautions	<ul style="list-style-type: none"> For external use only. May be used for cleansing the face but avoid direct contact to the eye. Can be used for incontinence clean up but recommend using products intended for this purpose (e.g. Deodorant Clean-Up Cloths, Comfort Shield Barrier Cream Cloths or Remedy Foam Cleanser). Do not use to remove Extra Protective Cream, use Remedy Foam Cleanser. Do not flush cloths or put in macerator. Dispose in garbage.
Contraindications	<ul style="list-style-type: none"> Do not use with known allergy or sensitivity to any of the product ingredients.
Formats & Sizes	<ul style="list-style-type: none"> Package of 8 (20 x 20 cm) cloths Package of 5 (20 x 20 cm) cloths <div style="text-align: center; margin-top: 10px;">  </div>
Application Directions	
Package is single patient use.	To minimize risk of cross contamination.
Choose size of package based on client need (e.g., package of 5 for a lighter bath).	To minimize wastage.
To Apply	
<p>Use at room temperature or warm in a Sage warmer. Only use microwave for warming if facility/unit policy allows.</p> <p>Sage warmer's display system switch, found on back of warmer, should be set to "C" only if CHG Antiseptic Body Cleanser wipes are in warmer. If no CHG Antiseptic Body Cleanser wipes in warmer, set display system switch to "B". The coloured lights indicate the heating status of the package adjacent to the light:</p> <ul style="list-style-type: none"> Yellow light – package has not reached the target temperature. Steady green light – package has reached target temperature. Flashing green light – package has been at target temperature for 48 hours and should be used first. Red light (only visible if display system switch is set to "C") – package has been at target temperature for over 84 hours and CHG Antiseptic Body Cleanser wipes should be discarded and other wipes should be used first. <p>Once package has been removed from warmer and taken to client's bedside, do not put back in warmer for re-warming.</p>	<p>Warm cloths provide increased client comfort.</p> <p>All Sage products can be in the warmer with display system switch set to "C" as this does not impact temperature.</p> <p>Packages take approximately 2 hours to warm.</p> <div style="text-align: center; margin: 10px 0;">  </div> <p>All wipes except CHG Antiseptic Body Cleanser Cloths are safe to use after 84 hours at target temperature.</p> <p>To reduce the risk of cross-contamination between clients.</p>



Skin and Wound Product Information Sheet

<p>Warming in microwave is not recommended. Only use microwave for warming if facility/unit policy allows.</p> <ul style="list-style-type: none"> ● Peel open label and place in microwave. <ul style="list-style-type: none"> ○ Package of 8: heat no longer than 30 seconds ○ Package of 5: heat no longer than 18 seconds <p>Closely monitor heat level, especially for infants or unresponsive or skin-sensitive individuals. Do not use if excessively hot.</p>	<p>Inconsistent heating can occur. Microwave wattage varies. Adjust heating time accordingly.</p> <p>Use caution when determining heat level as gloves diminish sensitivity to heat.</p>
<p>Use 1 washcloth for each area of body as described below.</p> <p>For package of 8 cloths:</p> <ol style="list-style-type: none"> 1. Face, neck and chest 2. Right arm 3. Left arm 4. Perineum 5. Right leg 6. Left leg 7. Back 8. Buttocks <p>For package of 5 cloths:</p> <ol style="list-style-type: none"> 1. Face, neck, chest 2. Arms 3. Perineum 4. Legs 5. Back, then buttocks <p>Do not rinse.</p>	<p>To reduce the spread of microorganisms.</p> 
<p>Disposal</p>	
<p>Discard each cloth into garbage after use. Do not flush cloths or put in macerator.</p>	<p>Cloths may plug/clog plumbing.</p>
<p>Expected Outcome</p>	
<p>Clean moisturized skin. Product performs as expected.</p>	<p>If product does not perform as expected, notify NSWOC/Wound Clinician and consider submitting a Product Concern form.</p>
<p style="text-align: center;">For further information, please contact: NSWOC or Wound Clinician</p>	