



## Skin and Wound Product Information Sheet

| InView Special   |   |
|--|---|
| <b>Classification</b>  | <b>Urinary Collection Device/Male: Condom/Short</b>   |
| <b>Key Points</b>  | <ul style="list-style-type: none"> <li>• Self-adhering, breathable, clear silicone (non-latex) condom with a plastic disposable cap to maintain adhesive integrity in package and a reinforced Funnel Tip (resists kinking and twisting to keep urine pathway open) which contains and collects urine.</li> <li>• Prevents Incontinent Associated Dermatitis by preventing urine being in contact with the skin.</li> <li>• Helps contain odour.</li> <li>• Improve patient dignity.</li> <li>• Facilitates the measurement of urinary output.</li> </ul> |
| <b>Indications</b>   | <ul style="list-style-type: none"> <li>• Used for male patients with less than (&lt;) 4cm of penile shaft in resting state</li> </ul>   |
| <b>Precautions</b>   | <ul style="list-style-type: none"> <li>• Does not adhere to skin if hair is present.</li> <li>• Carefully remove condom to prevent skin epidermal stripping.</li> <li>• Use sizing guide to obtain appropriate sized device.</li> </ul>   |
| <b>Contraindications</b>   | <ul style="list-style-type: none"> <li>• Do not use on irritated or broken skin.</li> <li>• Do not use oils, lotions, creams, emollients or powders under the condom.</li> <li>• Do not use if patient has allergy or sensitivity to silicone.</li> </ul>   |
| <b>Formats &amp; Sizes</b>   | <div style="display: flex; align-items: flex-start;"> <div style="flex: 1;"> <p>Sizes</p> <ul style="list-style-type: none"> <li>• 25mm</li> <li>• 29mm</li> <li>• 32mm</li> <li>• 36mm</li> <li>• 41mm</li> </ul> <p>Sizing guide available in Stores or <a href="#">click here</a></p> </div> <div style="flex: 1; text-align: center;"> </div> <div style="flex: 1; text-align: center;"> </div> </div>  |
| Application Directions   |   |
| Preparation  | Rationale   |
| <p>Measure the shaft (length) of the penis.</p> <p>Use the color-coded measuring/sizing guide at base of penis to determine correct size of condom needed. If patient is between sizes choose the larger size.</p> <p>Clean and thoroughly dry peri-genital skin and penis. Ensure foreskin is pulled back into correct anatomical (neutral) position.</p> <p>Trim any pubic hair if necessary.</p>  | <p>If penis length is 4cm (≥) or greater then use InView Special condom catheter.</p> <p>Applying a size to small may result in skin damage to penis. If a size is too large may result in leakage.</p> <p>Oils, lotions, creams, emollients and powders will interfere with adhesion.</p> <p>Hair will interfere with maximum adherence to skin and may make application and removal of device painful.</p>  |
| Application of Condom  |   |
| <p>Remove the protective cap from the condom catheter.</p> <p>Pinch the condom catheter closed where the funnel narrows.</p> <p>Touch the closed end of the condom catheter to the tip of the penis while holding the shaft of the penis securely with the other hand.</p> <p>Push the rolled band off the collar of the condom catheter. Holding the condom catheter at the tip of the penis, continue rolling down the length of the penis.</p> <p>Gently squeeze condom catheter to penis to ensure there is full contact of the adhesive of the condom catheter to the penis.</p> <p>Attach external urinary condom catheter to drainage bag.</p> <p>Inspect condom catheter 5-10 minutes after application and gently squeeze condom catheter to penis.</p> | <p>Correct placement is important for good wear time. If the condom catheter is placed too far forward, the tip of the catheter may twist, obstructing the free flow of urine. If the catheter is placed too far back on the penis, the urine may not be able to funnel down quickly therefore leading to back-up and leakage.</p> <p>To ensure adequate adhesion and minimize leakage.</p> <p>To ensure no leakage due to change in penis size during application.</p>   |



## Skin and Wound Product Information Sheet

|  |  |
|--|--|
| <b>Daily Care</b>  |  |
| <p>Check catheter for leakage every 2 hours</p> <p>Change external urinary condom catheter every 24 hours.</p> <p>Use warm wash cloth, held over the catheter for approximately 30-60 seconds. Remove by gently rolling the condom catheter off the skin. If this does not work may use a no sting adhesive remover wipe.</p> <p>Wash and dry peri-genital skin and penis after removal and before new application</p> | <p>Warm water will help soften the adhesive to make removal easier</p> |
| <b>Expected Outcome</b>  |  |
| <p>Peri-genital skin remains intact.</p> <p>Urine volume is measured.</p>  | <p>If condom does not work, then consider a male urinary pouch.</p>    |
| <p><b>For further information, please contact your Wound Clinician or ET/WOC Nurse</b></p>   |  |