



## Product Information Sheet

Jelonet	
<b>Classification</b>	<b>Non-Adherent Contact Layer: Paraffin</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• Conformable, paraffin gauze dressing.</li> <li>• Allows the passage of wound exudate into secondary dressing.</li> <li>• Prevents wound bed trauma by decreasing adherence of secondary dressing.</li> <li>• Primary dressing requiring a secondary dressing.</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• For prevention of wound bed trauma.</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• When used on burns treated with meshed grafts or after facial resurfacing, imprinting of dressing can occur if too much pressure is applied at the site.</li> <li>• Use with caution in areas of undermining and do not use in sinus tracts due to potential of fraying.</li> <li>• Has not been evaluated on pregnant/lactating individuals or neonates/infants, consult with physician/NP prior to using on these populations.</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Sensitivity or allergy to paraffin or any other components of the dressing.</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>• Sheets:                             <ul style="list-style-type: none"> <li>○ 5 x 5 cm</li> <li>○ 10 x 10 cm</li> <li>○ 10 x 40 cm</li> </ul> </li> </ul> 

Directions	Rationale / Key Points
<b>Selection</b>	
<p>Choose a size that just covers the wound. Jelonet can be cut to fit the wound size.</p> <p>For a wound with depth, choose appropriate wound filler for amount of exudate expected and the anticipated frequency of dressing change.</p> <p>Choose secondary dressing based on amount of wound exudate expected and the anticipated frequency of dressing change.</p>	<p>Overlap onto surrounding skin can cause maceration.</p> <p>Refer to <a href="#">Wound Packing Procedure</a> or QR Code below</p>
<b>Preparation</b>	
<p>Cleanse wound and periwound / surrounding skin with sterile normal saline or agency approved wound cleanser.</p> <p>Dry periwound / surrounding skin.</p> <p>If required and appropriate for secondary dressing, apply barrier film to periwound skin. Refer to Product Information Sheet for secondary dressing to determine if barrier film is appropriate.</p>	<p>See <a href="#">Wound Cleansing Procedure</a> or QR Code below.</p> <p>To protect periwound skin from moisture associated skin damage and medical adhesive related skin injury. Barrier film may interfere with the function of some cover dressings, (e.g., some silicone dressings).</p>
<b>Application</b>	
<p>Remove both backing papers from the dressing.</p> <p>For wounds with minimal depth (less than 1 cm): cover wound bed with single layer of Jelonet.</p> <p>For wounds with depth (more than 1 cm) or undermining: cover wound bed with single layer of Jelonet. Then lightly fill dead space up to skin level with appropriate wound filler.</p>	<p>Handle dressing by edges only. Excess jelly will remain on backing papers.</p> <p>Multiple layers may restrict the exudate from going into the secondary dressing leading to a wet wound bed.</p> <p>Use with caution in areas of undermining. Do not use in sinus tracts. Multiple layers may restrict the exudate from going into the wound filler. Refer to <a href="#">Wound</a></p>



## Product Information Sheet

Directions	Rationale / Key Points
Apply secondary dressing to cover the wound.	<a href="#">Packing Procedure</a> or QR Code below.
<b>Removal</b>	
<p>Consider using adhesive remover to remove adhesives (e.g. border dressings, tape).</p> <p>Gently lift the edge of the secondary dressing remove.</p> <p>Remove wound filler (if present).</p> <p>Gently lift the edge of the Jelonet and remove from wound.</p> <p>If Jelonet has been cut, ensure all loose fibres are removed from wound bed.</p>	<p>To decrease risk of medical adhesive related skin injury (MARS).</p> <p>Jelonet should not adhere to wound bed and should lift with no trauma.</p> <p>Dressing may fray when cut.</p>
<b>Frequency of Dressing Change</b>	
<p>Usually changed daily but can be left in place until dressing starts to dry out.</p> <p>Secondary dressing can be changed as needed while leaving Jelonet in place provided Jelonet is still moist.</p>	<p>Dry dressing may adhere to wound bed.</p> <p>Secondary dressing change frequency is dependent on amount of wound exudate.</p>
<b>Expected Outcomes</b>	
<p>Jelonet, wound filler and secondary dressing do not adhere to wound bed.</p> <p>Product performs as expected.</p>	<p>If product does not perform as expected, notify NSWOC/Wound Clinician and then consider submitting a <a href="#">Supply Chain Product Concern Form</a>.</p>
<b>QR Codes</b>	
<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p><b>Wound Packing</b></p> </div> <div style="text-align: center;">  <p><b>Wound Cleansing</b></p> </div> </div>	<p style="text-align: center;"><b>For further information please contact NSWOC/Wound Clinician</b></p>