



## Skin and Wound Product Information Sheet

<b>No-Sting Skin Prep</b>	
<b>Classification</b>	<b>Skin Barrier: Film</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• Liquid protective film; provides transparent, waterproof, breathable protective layer</li> <li>• Alcohol-free, latex-free, fragrance-free, preservative-free</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• For protection of intact or damaged skin from wound exudate, ostomy drainage, incontinent episodes; tube/drain drainage, adhesive-type dressings and friction</li> <li>• For sealing ostomy or medicated powders to skin surface (crusting application)</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• Product is flammable until dry</li> <li>• Keep out of reach of children</li> <li>• Avoid contact with eyes</li> <li>• If redness or other signs of irritation appear discontinue use</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Do not use for clients with known sensitive to any of the ingredients</li> <li>• Do not apply directly to any open wounds</li> <li>• Do not use for premature infants or infants younger than one month old</li> <li>• Do not use with Molnlycke silicone dressings</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>• Wipe                             <ul style="list-style-type: none"> <li>▪ 1 ml</li> </ul> </li> <li>• Swab stick                             <ul style="list-style-type: none"> <li>▪ 3 ml</li> </ul> </li> <li>• Spray bottle                             <ul style="list-style-type: none"> <li>▪ 28 ml</li> </ul> </li> </ul> <div style="text-align: center;"> </div>
<b>Application Directions</b>	
When using the Spray Bottle product, label with client name and date of opening.  Skin should be clean and dry prior to application.	<b>Rationale</b>
Product is single client use and may be used up until its expiry date.	
<b>To Apply</b>	
Apply a single uniform coating of No-Sting Skin Prep over the entire area of concern. If a section of the area is missed or a second application is required, then reapply only when the first application has thoroughly dried (30 seconds).	If not allowed to dry, the barrier film will not 'set-up' properly.
If applied to an area with skin folds or other skin to skin contact, ensure that areas are kept separated until the film has dried.	If skin surface touch each other before drying the fold/area will stick together.
Allow film to thoroughly dry (30 seconds) before applying tape, dressing or device.	
<b>To Remove</b>	
Removal of the film is not necessary prior to reapplication.	Removal of the film is not necessary unless build-up occurs (white flakey, dry areas where film has been applied) use an adhesive remover if this occurs.
<b>Frequency of Reapplication</b>	
Provides an effective film barrier for up to 4 days when used alone or under non-adhesive dressings.  When used under adhesive dressings, reapplication is necessary each time the dressing is changed as the film can be removed by the adhesive.  For Incontinent Associated Dermatitis: when used for protection or used with non-adhesive products, reapplication is recommended: <ul style="list-style-type: none"> <li>• every 24 -96 hours, depending on frequency of cleansing</li> <li>• every 12-24 hours, may be necessary for extreme cases; e.g., constant diarrhea.</li> </ul> Removal of the film barrier is not required prior to reapplication.	



## Skin and Wound Product Information Sheet

<b>Crusting Application</b>	
Apply a dusting of ostomy or medicated powder to area of concern. Spray or dab (using wipe) No-Sting Skin Prep over the powder and allow to dry.	Powder will adhere to moist areas. The film will seal in the powder and act as a cover dressing.
If the area is weepy, repeat application of powder and film to a maximum of 3 times; allow for the film to dry between applications.	
<b>For further information, please contact your Wound Clinician.</b>	