


QuickChange Male Penile Wrap Application for a Hidden/Covered Penis

Step 1: Place the 1st pad


- Apply skin protectant to area as per care plan.
- If needed, lift abdominal skin fold up to see peri-area.
- Slide pad around scrotum with:
 - White side up, round edge towards the feet, tuck the wrap between the thighs.
 - Pointed flap and
 - Straight flap towards the head.

Note: straight flap has hook fastener, fold over the top of flap to hide this.

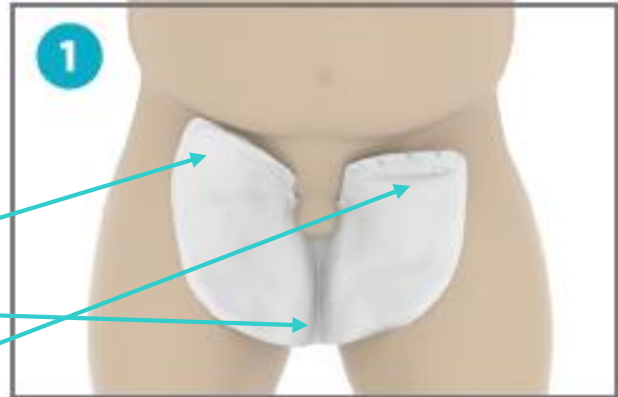

Step 2. Place the 2nd pad

- Place white side up over the belly.
- Position the wrap to ensure the penis opening is visible.
- Fold over the edge of straight flap to cover the fastener.

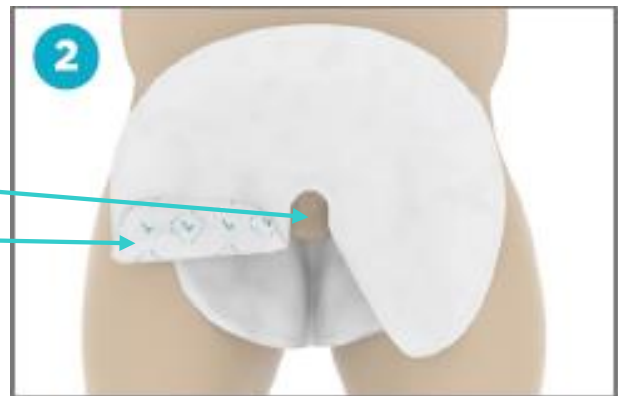

Step 3. Position the pad

- Fold 2nd wrap in half lengthwise and cover the penis opening.
- If needed, lay the abdominal skin fold back down over the peri-area.

