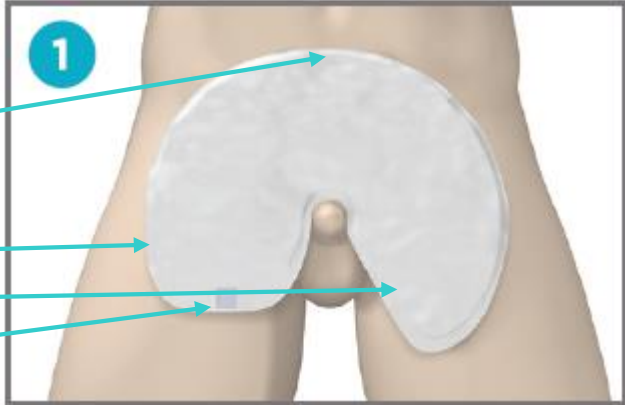
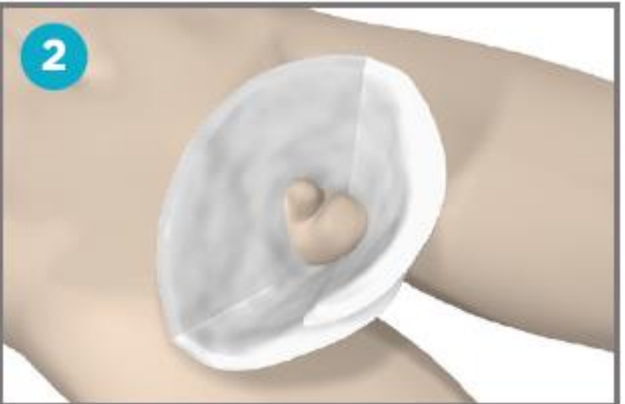


## QuickChange Male Penile Wrap

### Application for Scrotal Edema

<p><b>Step 1: Place the pad</b></p> <ul style="list-style-type: none"><li>• Apply skin protectant to area as per care plan.</li><li>• Slide pad around the penis with:<ul style="list-style-type: none"><li>○ White side up, round edge towards the client's head</li><li>○ Straight flap and</li><li>○ Pointed flap towards the legs.</li></ul></li></ul> <p>Note: straight flap has hook fastener.</p>	
<p><b>Step 2. Surround the penis/scrotum</b></p> <ul style="list-style-type: none"><li>• Slide pointed flap underneath the scrotum.</li><li>• Pull the straight flap over and position under the straight flap to create a bowl to cradle the edematous scrotum.</li></ul>	
<p><b>Step 3. Secure the wrap</b></p> <ul style="list-style-type: none"><li>• Using two fingers, pinch the hook fastener and inner pointed flap together.</li></ul>	