


QuickChange Male Penile Wrap Application for a Small or Retracted Penis

Step 1: Place the pad


- Apply skin protectant to area as per care plan.
- Slide pad around the penis with:
 - White side up, round edge towards the head
 - Straight flap and
 - Pointed flap towards the legs.

Note: straight flap has hook fastener.

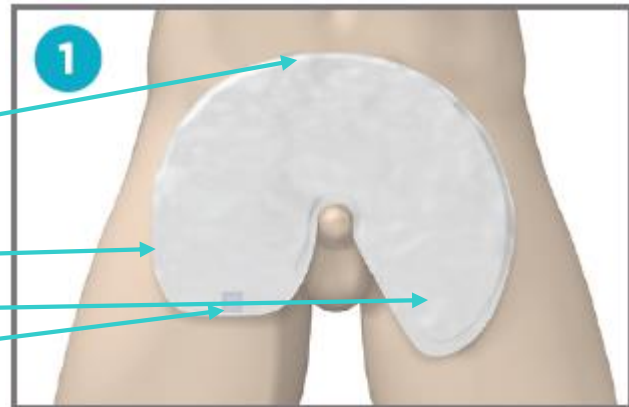

Step 2. Surround the penis/scrotum

- Slide pointed flap underneath the scrotum.
- Lift flap and scoop scrotum forward.
- Pull the straight flap over and under the pointed flap.

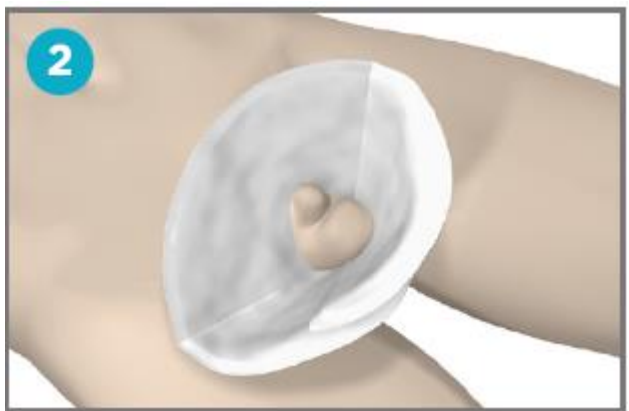

Step 3. Secure the wrap

- Using two fingers pinch the hook fastener and inner pointed flap together.
- If possible, lay the wrap on the lower abdomen.

