


QuickChange Male Penile Wrap Application for Standard or Long Penis

Step 1: Place the pad


- Apply skin protectant to area as per care plan.
- Position the penis on the lower abdomen with penis facing straight towards the client's head.
- Slide pad under the penis with:
 - White side up, round edge towards the head
 - Straight flap and
 - Pointed flap towards the legs.

Note: straight flap has hook fastener.

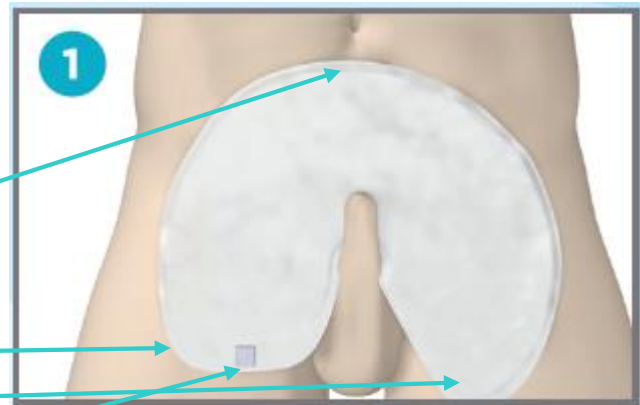

Step 2. Cover the penis

- Fold the pointed flap over top of the penis at a 45° angle.
- Bring the flap edge up to meet the rounded top edge.

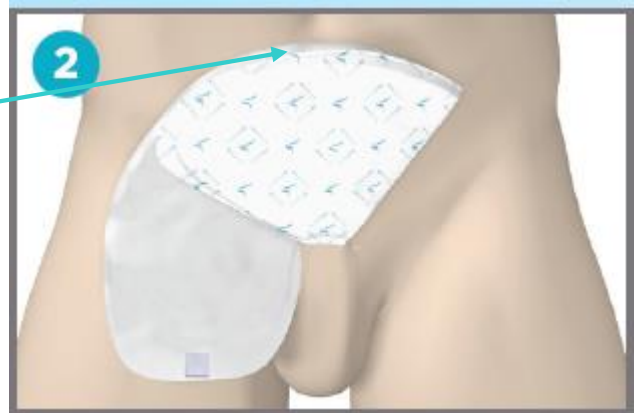

Step 3. Secure the wrap

- Fold the remaining flap up over the wrap; slightly pull the flap upward to align its edge with the round edge.
- Using two fingers, pinch the hook fastener and inner pointed flap together.

