

Tubigrip Lower Leg Application for Compression		
Tubigrip Size	Calf Circumference Minimum - Maximum Centimetres	Sub-Bandage Compression
A	10.0 – 14.5 cm	20 – 30 mm/Hg
B	14.5 – 29.0 cm	10 – 20 mm/Hg
C	22.0 – 35.0 cm	
D	26.0 – 40.0 cm	
E	29.0 – 46.0 cm	
F	35.0 – 53.0 cm	
G	40.0 – 62.0 cm	5 – 15 mm/Hg
J	54.0 – 82.0 cm	

Example of sizing & compression

If calf measures 14.5cm, use Size A to provide 30mm/Hg of compression or use Size B for 10mm/Hg.

If calf measures 25cm, use a Size B stocking to provide compression closer to 20mm/Hg; use a Size C stocking to provide compression closer to 10mm/Hg.